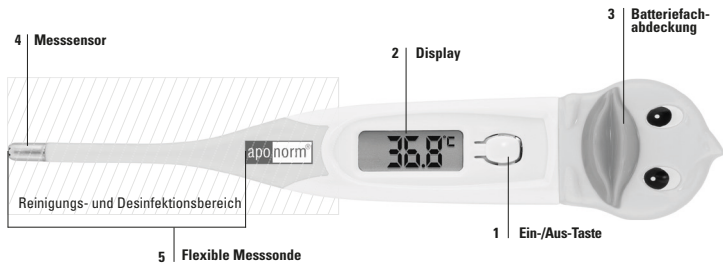
THERMOMETER FLEXIBLE ENTE

PZN-10545841
Bestell-Nr. 047234

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technology by
microlife

art.-no. MT700

DEUTSCH

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- ① Ein-/Aus-Taste
- ② Display
- ③ Batteriefachabdeckung
- ④ Messsensor
- ⑤ Flexible Messsonde

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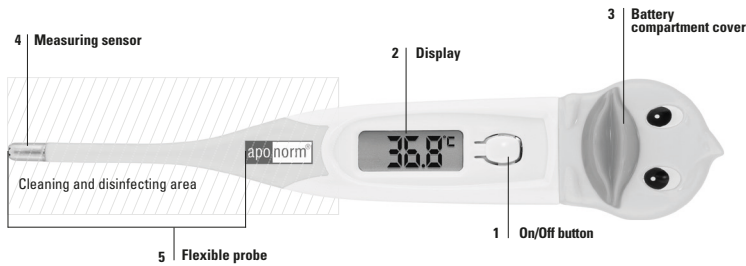
ENGLISH

1. Device overview

- ① On/Off button
- ② Display
- ③ Battery compartment cover
- ④ Measuring sensor
- ⑤ Flexible probe

2. Safety Instructions

- Follow these instructions for use. This document contains important information about the operation and safety of this device. Please read this document carefully before using the device and keep it for future reference.
- This device may only be used to measure human body temperature (in the mouth, rectum or armpit). Do not attempt to use the device in any other measurement location, such as in the ear, as this could lead to incorrect readings and/or injury.
- Do not use the device if you notice any damage or if you notice anything unusual.
- For personal hygiene, we recommend cleaning the appliance – before first use – according to the «Cleaning and disinfection» instructions below.
- The minimum measurement period must be observed without exception until you hear the signal tone! Depending on the measurement location, it may be necessary to continue measuring after the signal tone, see section «Measurement method/normal body temperature».
- Make sure that children do not use the appliance unattended; some parts are so small that they could be swallowed.
- Do not use the device near strong electromagnetic fields such as mobile phones or radio equipment. Keep a minimum distance of 3.3 m from such devices when using this device.
- Protect the device from knocks and blows!
- Avoid bending the thermometer measuring probe by more than 45°!
- Avoid ambient temperatures above 60 °C. NEVER boil the appliance!
- To clean the device, use only the disinfectants specified under «cleaning and disinfection».
- We recommend that an accuracy test is carried out by an authorised laboratory for the instrument every 2 years or after a mechanical impact (e.g. shock from falling).

⚠ ATTENTION: The measurement result displayed by this device is not a diagnosis! Do not rely solely on the measurement result.

♻ Batteries and electronic devices must not be disposed of in household waste but must be disposed of in accordance with local regulations.

☔ Protect from moisture.

⚠ Type BF application part

📖 Study the manual carefully before use.

3. Preparing the thermometer for use

To switch on the thermometer, press the On/Off button ①; a long signal tone indicates that the thermometer is switched on. First a display test is carried out – all segments should be displayed. Then, if the ambient temperature is less than 32 °C, an «---» ("low") and a flashing «°C» appears on the display ②. The thermometer is now ready for measuring the temperature.

4. Functional test

Every time the thermometer is switched on, a function test is carried out automatically. In the event of a malfunction (inaccurate measurement), «ERR» appears in the display ②. In this case the device can no longer be used. Please contact your pharmacy for replacement (warranty check).

5. Using the thermometer

Select your preferred measuring method. During the measurement, the current temperature is continuously displayed and the «°C» symbol flashes. If the signal tone sounds and the «°C» stops flashing, this means that the measurement is finished, and the thermometer is ready for reading. Depending on the measuring method, it could be necessary to continue the measurement longer than is actual shows by the signal tone. Please refer «Measuring method/normal body temperature». The thermometer can be switched off directly by briefly pressing the On/Off button ① – this extends battery life. Otherwise the thermometer switches off automatically after approx. 10 minutes.

6. Display the last measured value

When turning on ① the thermometer, the last measurement will be displayed with a «M» („Memory”).

7. Measuring method/normal body temperature*

7.1. Under the arm (axillary) / 34.7 - 37.3 °C (Adult) & 35.2 - 37.8 °C (Child)

Wipe the armpit with a dry cloth. Place the measuring sensor ④ of the thermometer as far up in the armpit as possible and clamp it to your body with your arm. This ensures that the room air does not affect the measured value. Since the armpit takes longer to reach its stable temperature, wait **at least 5 minutes** regardless of the beep.

7.2. In the mouth cavity (oral) / 35.5 - 37.5 °C (Adult) & 36.0 - 38.2 °C (Child)

Do not eat or drink anything warm or very cold 10 minutes before the measurement. The mouth should be closed at least 2 minutes before starting a measurement. Place the thermometer with the measuring sensor ④ into one of the two tongue pockets, directly at the root of the tongue. The measuring sensor should be in good contact with the tissue. Press your lips firmly together and hold your mouth closed, so that the measurement result is not influenced by the air you are breathing. Breathe calmly through your nose during the entire measurement process. If this is not possible due to obstructed airways, another measuring method should be used.

Minimum measurement time: 30 seconds!

7.3. In the anus (rectal) / 36.6 - 38.0 °C (Adult) & 37.0 - 38.4 °C (Child)

For rectal measurement, it is best to lie on your back or on your side and bend your legs. To avoid injuries, it is recommended to additionally grease the measuring tip (e.g. with Vaseline). The measuring sensor ④ of the thermometer is carefully inserted 2-3 cm into the rectum/anus. If you are not sure about this method of measurement, you should consult a specialist for instruction/training.

Minimum measurement time: 30 seconds!

*Don't be too strict about keeping within the exact normal temperature ranges – rather consider them as guidelines that should be in line with your physical condition.

8. Cleaning and disinfection

For disinfecting the thermometer at home, use an isopropyl alcohol swab (70%) or a cotton cloth moistened with 70% isopropyl alcohol to wipe off coarse, visible surface dirt from the thermometer measuring probe ④ (= complete cross-hatched area). Note: Please observe the application and safety instructions of the disinfectant manufacturer. Then immerse the thermometer measuring probe ④ (= complete cross-hatched area) in 70% isopropyl alcohol for at least 5 minutes (max. 24 hours). Let the disinfectant dry for 1 minute before using the thermometer again. Avoid immersing or wiping the display ② to protect it from damage.

9. Battery replacement

As soon as the «▽» symbol appears on the right of the display ②, the battery is exhausted, and it must be changed. To do this, remove the cover of the battery compartment ③ and replace the exhausted battery with the «+» sign pointing upwards. Make sure that the same type of battery is used. Batteries are available in every electrical shop. For disposal, please observe the instructions under «Safety instructions».

10. Technical specification

Type	Maximum thermometer
Measuring range	32.0 °C to 43.9 °C Temp. < 32.0 °C: Display «---» for "low" (too low) Temp. > 43.9 °C: Display «H» for "high" (too high)
Measurement accuracy	± 0.1 °C between 34.0 to 42.0 °C ± 0.2 °C between 32.0 to 33.9 °C and 42.1 to 43.9 °C
Operating conditions	10.0 °C to 40.0 °C
Self-test	Automatic internal check for 37.0 °C test value; «ERR» (error) is displayed if the deviation is > 0.1 °C
Display	Liquid crystal display (LCD) with three digits; smallest display unit = 0.1 °C
Memory	For automatically storing the value measured last
Storage conditions	Temperature: - 25.0 °C to + 60.0 °C, Air humidity: 15% to 95% (without condensation)
Battery	LR41 (1.5V) / SR41 (1.55V)
Battery life	approx. 4,500 measurements (with new battery)
IP class	IP22
Standard	EN 12470-3, clinical thermometer; ASTM E1112; IEC 60601-1; IEC 60601-1-2 (EMC); IEC 60601-1-11
Electromagnetic compatibility	Device complies with the specifications of the IEC 60601-1-2 standard
Average lifetime	5 years or 10,000 measurements

11. Guarantee

This device is guaranteed for 30 years from the date of purchase. The guarantee is only valid if a sales receipt issued by the dealer is presented. Mailing costs for sending the thermometer in by the customer will not be reimbursed. For customers outside Germany, the processing of the guarantee case also depends on the reimbursement in advance of the costs required for the return shipment.

The legal rights of the consumer (e.g. warranty rights) are not limited by this guarantee and remain unaffected by it.

• The guarantee only covers the thermometer itself. Batteries and packaging are not covered by the guarantee.

• If the device is opened or if changes are made to it, the guarantee becomes void.

• The guarantee does not cover damage caused by improper handling, discharged batteries, accidents or failure to follow the instructions for use.

Please always contact your specialist dealer/pharmacy or our apornorm® customer service (info@apornorm.de)

You can find detailed information about our thermometers, blood pressure monitors, inhalers and our services at www.apornorm.de

Date of purchase:

Serious incidents must be reported to the manufacturer and the proper authority of the member state in which the user and/or patient is established.

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GmbH & Co KG
Microlife AG